C7581

CAMILLA ELÉCTRICA / ELECTRIC TABLE / TABLE ÉLECTRIQUE

INSTRUCCIONES - INSTRUCTIONS

- * Leer antes de utilizar por primera vez.
- * Read carefully before the first use.
- * A lire avant la première utilisation.

etiqueta CE

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INSTRUCCIONES

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ELECTRIC TABLE INSTRUCTIONS

Ecopostural thanks you for buying our products. The item you have just purchased has been manufactured according to high quality and security standard. Follow the below-mentioned instructions carefully in order to be able to make use of it for a long time. Click on to the TECHNICAL INFORMATION section in our website www.ecopostural.com

*** Our company will decline any responsibility for any damage occurred if ignoring these instructions.**

1) GENERAL INSTRUCTIONS

- Please read carefully the operating and safety instructions before using our products. These instructions must be adhered to. Keep the instructions booklet for further check-ups.
- Familiarize yourself with the precise use of the item.
- If the table is handled by unqualified people in an inappropriate way or not according to the intended purpose, the use of the examination table may be dangerous. Be careful when handling the table as there is a risk of being trapped by the mobile parts and because of sudden movements.
- Regularly check the table entirely to find any sign of malfunction or damage. Periodically check mains cable and plug. We recommend to handle them with the utmost care and to avoid the bending of the cables.
- Worn or damaged pieces must be solely replaced by original pieces by the manufacturer depending on the table model.
- All components parts or accessories should be fitted or fixed in according to the special manufacturer's instructions. Any spare / replacement component part different to the manufacturer's could be dangerous for the people involved in the table handling.

*** Notice: The use of non original pieces or the alteration of the item will incur in a loss of validity of the manufacturer's guarantee, instructions and Declaration of Conformity.**

2) OPERATING INSTRUCTIONS

a) Installing

The table must lean on a solid and level ground.

The table is supplied with levellers placed in two of the legs. To level the table, simply wind or unwind them.

Notice: in certain floor surfaces the table legs might leave marks after a prolonged contact. Check the technical specifications of the floor manufacturer to avoid any trouble.

In order to move the table around, you will need to release the castors by pressing the locking levers downwards. To lock the castors, simply lift both locking levers upwards with your foot. These castor wheels are only suitable for moving the table around when required (maintenance...), NOT intended for patient transportation.

*** Caution: Before the patient lies down make sure the castors are locked.**

b) Load capacity

This item has been tested to support a **MAXIMUM STATIC WEIGHT of 175 Kg** evenly distributed all over.

* In case this weight is exceeded a safety device will automatically start-up. As a result the table might gradually move downwards regardless of the motor activation. Reduce the load in order to restore the proper operation of the table.

Absolutely avoid leaning on the very edge for support as it may tilt over.

If pressure on a reduced area exceeds the allowed weight, the table upholstery could split up. Keep the patients from getting on by their knees.

To **get on** the table the patient must seat at the middle of the table, raise his legs and lean on his side. Then, he can lie down. Weight must necessarily be distributed evenly and not concentrated on either side of the table.

When **getting down** the table different procedures apply depending on the position of the patient. Therefore, if he is lying down (prone position) he has to slide gently towards the middle of the table, lean on his side and go up by standing on his arms. He will then be able to put his legs out by the table's side and easily stand up.

If the patient is lying upwards he will have to stand on his arms, get back to a seated position and then go down the table.

* **Caution:** Never use the head or back rest sections to lean forward as these parts are not aimed at supporting such a pressure.

c) Height adjustment

The central height of the table is operated by an **electric motor**.

The motor supplied in our electric tables is sent disconnected. Before plugging the table in, connect the air pipes according to the attached instructions.

To operate the supply cable must be plugged into a 220-230 V socket.

The height adjustment is done by smoothly pushing sideways (right/ left) the **foot rails** located around the base of the table.

* **Caution:** **DO NOT press down the foot rails; this is not a footswitch.**

The scissor-shaped tables may present a certain vibration, especially when going up or down without any load. This vibration does not mean the table to be abnormal.

* **Caution:** **Safely position patient before table height adjustment. Do not insert hands, feet or garments into the moving parts of the elevating mechanism as this could result in serious injuries.**

d) Sections adjustment

The **head-/backrest** section is operated by a gas spring (3)

The **central section** is operated by knob/lever (1)

The **feet section** is operated by rack (2)

(1) Knob/Lever: With one hand, loosen the knob or lever to release the bar. With the other hand, position the section at the required tilting. Release the knob or lever and the section will lock itself.

(2) Rack: Lift the section up to the required position and the rack will position itself automatically. To lower the section, first raise it up to a certain extend or completely (as appropriate) in order to release the rack.

* *The hinges on both sides must be properly fixed and positioned at the same height.*

(3) Gas spring: Press the mechanism situated at the very edge of the section towards the upholstery and the table section will go up slowly. Release mechanism to lock position. To lower the section, press the mechanism with one hand and with the other hand, push the section downwards to the required position.

* **Caution:** **Never adjust any of the table sections while the patient is leaning on.**

Safely position your fingers and hands away from the upholstery edge in order to avoid any accidental jamming.

How to adjust the articulated leg supports

To adjust the *height* of the leg support, loose the black push-button lever to unlock the bar and move it up or downwards to attain the required height or simply pull the bar out completely. Screw back on the lever to lock.

To adjust the *tilting* of the leg support, loose the black lever under the support upholstery to unlock the swivel plate. Next incline the leg support to the required position and screw back on the lever to lock.

By pressing the *red push button* of the lever you will be able to move the lever freely, no need to screw or

loose. This push button turns out to be very useful when the turning angle of the plastic levers is limited due the short distance to the table frame.

3) **MAINTENANCE**

*** Warning: For safety reasons, disconnect the table from the electrical supply before the table cleaning, repair or servicing.**

a) Metal frame

For a proper operation of the table, you should do the dusting regularly with a cloth.

We also recommend to periodically grease the axles at the moving parts with a liquid grease spray. Lay the couch on its side and take off the white caps protecting the nuts at the axles of the framework. Slightly unscrew the nuts and spray the product around the loosen nuts. Leave to settle in this position for 12 hours. Tighten the nuts and put the protective white caps back on. Lay the table on the other side and repeat.

Periodically check couch visually for any signs of damage and make sure that all bolts, nuts and other fasteners are tight.

Please note that the repeated exposure of the table to high levels of humidity is not recommended.

b) Electric motor

Keep this piece free from any dust or grease. Absolutely avoid spilling liquids on the motor because they could cause severe damage to the electric components despite the motor protection.

The handling of the electric motor is totally forbidden. Any manipulation will invalidate the guarantee.

A non-stop blinking operation of the foot rail or any other height adjustment mechanism could activate the thermal protection of the motor due to an overheating.

Examine the table periodically for any signs of damage or malfunction and contact immediately Ecopostural or the reseller for service/ spare parts. For further information, go to annex "Frequently asked questions about motors".

c) Section adjustment mechanisms

Keep the mechanisms free from any dust or grease.

The handling of the gas spring and/or the electric motor is totally forbidden. Any manipulation will invalidate the guarantee.

If any malfunction or failure is suspected, contact immediately Ecopostural or the reseller.

d) Upholstery

A wet cloth should be enough to do the dusting. Anyway, if the leather is not properly clean you can use a smooth sponge soaked with neutral soap water. Dry immediately afterwards. To prevent moisture from entering the seams, do not wash with abundant water.

It is highly recommended to use a towel cover like the A4459 or A4460 to protect the upholstery from sweat, massage creams or oils.

Use soapy water to clean the stains. **Avoid using alcohol, any solvent or stain remover** as the vinyl could be seriously damaged.

The use of infrared lamps might cause a cracking of the artificial leather. You should then put a cotton sheet or towel on the couch.

*** For further details please CHECK THE ENCLOSED UPHOLSTERY INSTRUCTIONS.**

ECOPOSTURAL GUARANTEE

All our equipment is covered by a **3 years** guarantee, from the original invoice date, against any manufacturing flaw or mechanical defect. It will be thus necessary to keep the invoice or any other purchasing document as proof of guarantee.

Wear and tear of the artificial leather will not be covered by the guarantee.

The following conditions will invalidate your guarantee: any damage or deterioration caused by misuse (e.g. inappropriate leather cleaning), by an undue unwrapping with a cutter (e.g. upholstery rips and tears) or by accident.

In those cases where the guarantee applies, Ecopostural will repair or replace the faulty part free of charge and return it at our expense within the EU territory.

For full details of the guarantee, please contact your supplier.

PRODUCT  **MANUFACTURED BY ECOPOSTURAL, S.L.U.** ESB12433991
Avda. Enrique Gimeno, 27 - 12006 Castellón de la Plana, **SPAIN**
www.ecopostural.com / info@ecopostural.com / +34 964 20 01 53

 The disposal of this product is subject to a specific EC Directive and it must be followed accordingly, as shown by the crossed skip symbol. Therefore, at the end of its useful life, you will have to apply to a specialized waste disposal centre.

Separate Collection of used products and packaging allows materials to be recycled and used again. Re-use of recycled materials helps prevent environmental pollution and reduces the demand for raw materials.

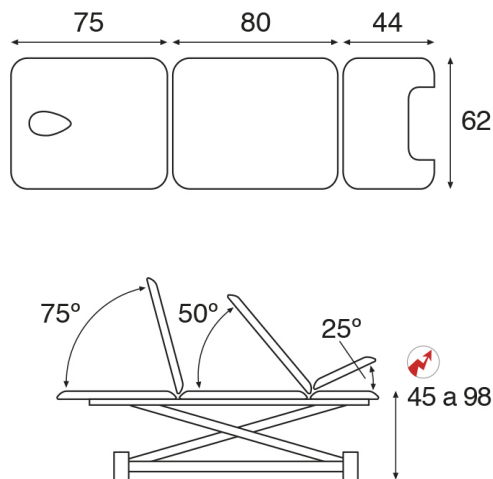
FREQUENTLY ASKED QUESTIONS ABOUT TYPE C, D, E, G, H, I MOTORS

You should notice that:

The motor is equipped with a THERMAL PROTECTION device. A non-stop operation of the motor may cause an overheating thus triggering an automatic disconnection. The motor won't break down but it will need a while to cool down before restoring its activity. Simply stop operating the motor when reaching the maximum extend and the thermal protection won't be activated.

The motor is pneumatically ACTIVATED by means of an air flow going through an air pipe. Therefore, any alteration in the air pipe or in the environmental conditions may affect the proper operation of the motor.

Problem:	Most common causes:	Solution:
The actuator (motor) is not moving. 	The actuator is not power supplied.	Check that the power point, the power cable and the different connections are properly plugged in
	The air pipe has been accidentally disconnected.	Connect the air pipe correctly.
	The air pipe has any holes or cuts, thus the air is leaking.	Replace the air pipe.
	The motor is hot. The thermal protection has been activated due to a non-stop operation of the motor or to a malfunction in the controls.	Unplug the table from mains. Wait about 15 min. until the motor cools off. If the problem persists, go to the section Controls not working properly'.
	The load is excessive.	Reduce the load. If the load is within the weight allowance and the actuator does not move anymore, contact your supplier (check first the whole troubleshooting).
The controls do not work properly or The motor is activated without previously operating the controls.  	The controls, the pipe and the actuator are exposed to sudden changes in temperature.	Level the pressure in the control system by disconnecting the air pipe, wait a minute and re-connect it.
	The controls, the pipe and the actuator are exposed to sudden changes in atmospheric pressure.	Level the pressure in the control system by disconnecting the air pipe, wait a minute and re-connect it.
	The air pipe is trampled or has any knots and bends.	Unwind the air pipe to undo the effect of any bend, knot... If the initial condition of the pipe is not restored it will have to be replaced.
	The electronic switch is stuck thus continuously activating the actuator.	Check that the foot rail system is in good condition. Contact your supplier for precise instructions.
	The control / switch is out of order.	Replace the control / switch.
The table moves downward without activating the motor.	The load is excessive.	Reduce the load. If the load is within the weight allowance and the tables keeps on going down, contact your supplier (check first the whole troubleshooting).

TECHNICAL FEATURES C7581**General characteristics**

Electric white scissor-shaped speciality table.

Standard upholstery dimensions: 62x200 cm (M47)

Optional upholstery dimensions: 70x200 cm (M67)

Recommended maximum weight: 175 Kg

Height variation: 45-98 cm

Electric motor-controlled height adjustable by side rails system.

Height adjustment without side misalignment

Steel frame with white epoxy powder coating heated at 250°C

70 mm diameter steel legs. With table levellers in two of the feet.

Retractable castors with foot lever. Just for moving the table around, NOT intended for patient transportation.

Detachable white paper roll holder included as standard.

Standard backrest type T13 - Square backrest. Oval breather hole and bung included.

Articulated leg supports (A4705) height adjustable and removable.

Stainless steel removable tray 15x30x4 cm.

Head-/Backrest adjustable by gas spring.

Central section adjustable manually, by knob/lever.

Feet section adjustable manually, by rack.

Approx. weight: 85 Kg

Upholstery Features

Fire-resistant Class M2 artificial leather, EC compliant.

Sanitized® anti-microbial and antibacterial protection.

Available in 22 colours.

Foam thickness 5 cm.

Foam firmness II - Soft

Electric Motor Features

Stroke length: 300 mm

Asynchronous motor 1-230V. AC $\pm 10\%$

Integrated thermal protection

Max. Power: 6000N - Average speed at load 0: 10mm/s

Voltage: 230V/50-60 Hz - Power consumption: 2A

Working cycle: 25s non stop / 400s off

Safety class: II - Standard protection: IP54

EMI (electromagnetic interference) Directive compliant

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