

Quick reference guide for updating touchecg with HD+

Package contents

1) Installation CD + memory stick USB



2) Acquisition device HD+ and patient cable




3) No. 2 AAA batteries



4) Alligator clip adapt. 10-piece set




5) ECG disposable electrode, tab, 100-piece package

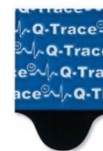

Note: This software version only works with the HD+ acquisition device; the clickecg model is not compatible.



1 – Removing the previous Touchecg

- Switch on the computer, make a **back-up copy** of the (C:\Cardioline\TouchECG\dbMonitor.mdb) patient database folder on an external disc;
- Open the PC Control Panel in the section for uninstalling a program;
- Select «Cardioline TouchECG» and click with the right mouse button, then choose «**Disinstalla**» (Uninstall);

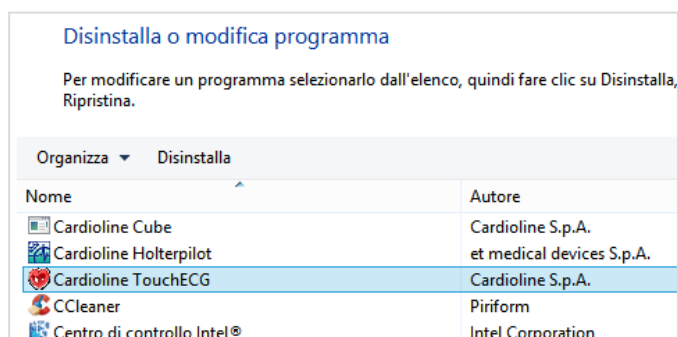


2 – Installation of the new program

- Insert the installation CD or USB memory key (1) provided;
- Open the «Setup/Setup Touchecg» folder;
- Start the «ToucheECG_setup.exe» program;
- If the program detects an existing installation folder of the program, confirm the use thereof;
- When the program detects the existing database, choose **NOT to overwrite it**;
- Follow the installation wizard until completion.

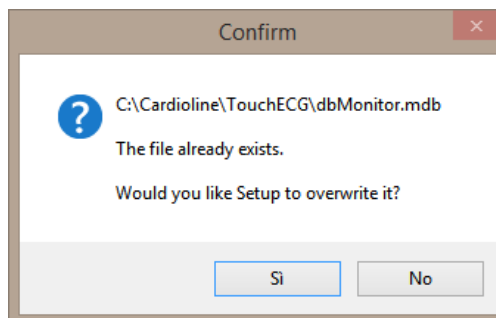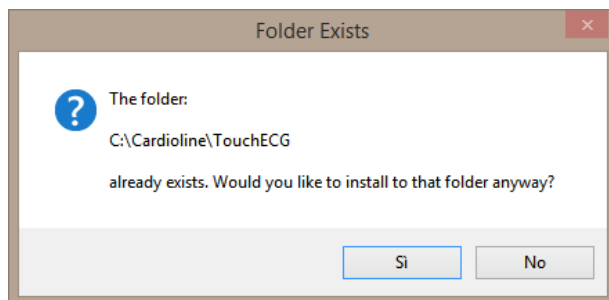


3 – Preparing the HD+ device


- Open the HD+ device battery cover and insert 2 batteries (A);
- Insert the patient cable into the corresponding socket (B);
- Switch on the HD+ device by pressing the power button (C);

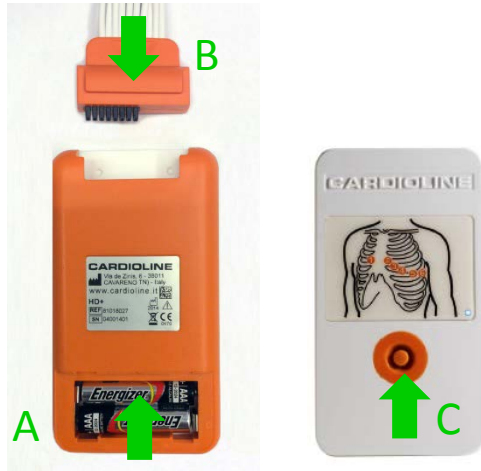

- Click on the Bluetooth icon on the PC and select «**Aggiungi un dispositivo Bluetooth**» (Add new Bluetooth device). Normally, the icon is found on the Windows taskbar or in the control panel;

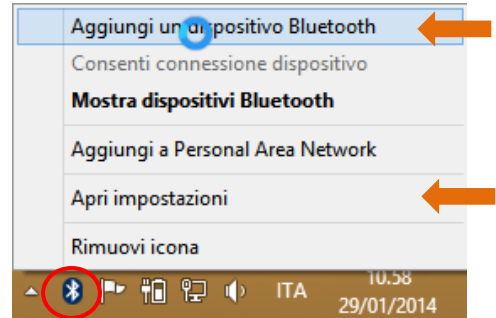

- Wait for Windows to complete the installation, then click on the Windows Bluetooth icon again and select «**Apri impostazioni**» (Open settings);
- Select the «Porte COM» (COM doors) window and take note of the **outgoing COM door**;

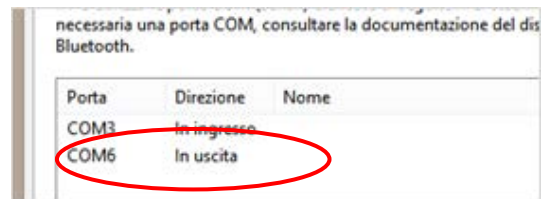

4 – Software setup

- Open the Touchecg program using the icon on the computer desktop;
- Click on the Settings button located on the bottom in the vertical bar and right of the screen;
- Open the «Unità ECG» (ECG unit) panel;
- Select the outgoing serial port obtained during the pairing procedure described in point (3);
- Press the «Imposta come predefinito» (Set as default) button and restart the program;

